# ONE COAT 7 UNIVERSAL

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EN

### Definition

ONE COAT 7 UNIVERSAL is a light-cured, one-component bonding agent used with the self-etching, selectively etching or total etch technique for adhesive restoration techniques.

Optionally, One Coat 7 UNIVERSAL can also be used with One Coat 7.0 Activator for chemically curing materials. Please follow the separate One Coat 7.0 Activator instructions for use.

### Composition

Methacrylates  
Photoinitiators  
Ethanol  
Water

### Areas of application

- Bonding in adhesive restoration techniques
- Adhesive attachment of composite materials and compomers on enamel and dentine
- Adhesive attachment of ceramic and composite restorations on enamel and dentine
- Composite material on ceramic, composite material, metal and amalgam
- Dentine sealing

### Application techniques

- Self-Etch: ONE COAT 7 UNIVERSAL is applied on enamel and dentine

- Selective-Etch: Enamel is etched, ONE COAT 7 UNIVERSAL is applied on enamel and dentine
- Total-Etch: Enamel and dentine is etched, ONE COAT 7 UNIVERSAL is applied on enamel and dentine

### Contraindications

Proven hypersensitivities to ingredients contained in ONE COAT 7 UNIVERSAL.

Inadequate oral hygiene. If it is impossible to keep the working area dry.

### Side effects

ONE COAT 7 UNIVERSAL ingredients can cause sensitivity in predisposed persons.

### Interactions

Phenolic and other substances (e.g. zinc oxide eugenol) which inhibit polymerisation must not be allowed to come into contact with ONE COAT 7 UNIVERSAL.

### Applications

#### Note

To protect against the bond curing, replace the cover on the mixing well immediately after dispensing.

#### 1. Conventional filling technique

##### 1.1 Preparing the cavity

According to the principle of the adhesive restoration technique. If indicated, a punctiform cap should be placed near the pulp using hard curing calcium hydroxide cements. Finish the enamel with a diamond bur. In case of uncut enamel, etch with phosphoric acid or clean with pumice. Then thoroughly rinse and dry the enamel. The use of a rubber dam (e.g. from HYGENIC/ROEKO) is recommended for hygienic reasons.

### 1.2 Conditioning

Apply phosphoric acid based etchant gel (e.g. Etchant Gel S) in accordance with the manufacturer's instructions for use with

- a) Selective enamel etching on enamel
- b) Total-Etch on enamel and dentine

Dry the cavity with a foam pellet or with oil-free compressed air.

**Important:** Do not let dentine dry out.

### 1.3 Application

#### From the bottle:

Shake the bottle well before use and dispense a drop into the mixing well. Close the bottle again immediately. Rub ONE COAT 7 UNIVERSAL into the cavity with a disposable dentalbrush for 20 s. Blow gently with oil-free compressed air for 5 s. Then light cure for 10 s (halogen or LED polymerization unit with light intensity >800mW/cm<sup>2</sup>). At lower light intensity the light curing is prolonged accordingly.

#### From the single dose:

Open the single dose immediately before use. Snap off the safety lock, rub ONE COAT 7 UNIVERSAL into the cavity with a disposable dentalbrush for 20 s. Blow gently with oil-free compressed air for 5 s. Then light cure for 10 s (halogen or LED polymerization unit with light intensity >800mW/cm<sup>2</sup>). At lower light intensity the light curing is pro-

longed accordingly. Avoid adhesives, as they could impair the accuracy of fit.

### 1.4 Composite/compomer filling material

Use the restoration material according to the manufacturer's specifications.

## **2. Fixation of inlays/onlays, veneers and crowns**

The use of ONE COAT 7 UNIVERSAL is only possible under the condition that sufficient incident light (according to the manufacturer's specifications) through the complete restoration is ensured.

### 2.1 Preparing the restoration

The restoration is pretreated depending on the material and according to the respective manufacturer's specifications.

### 2.2 Preparing the cavity

Proceed as in point 1.1

### 2.3 Conditioning

Proceed as in point 1.2

### 2.4 Application

Proceed as in point 1.3

### 2.5 Cementing

According to the respective manufacturer's specifications.

**Important:** Avoid over application of the adhesive as this could impact the fit of the restoration.

## **3. Bonding to other surfaces**

### 3.1 Preparation

According to the respective manufacturer's specifications.

#### 3.1.1 Ceramics

Intraoral application:

Roughen the ceramic surface with a diamond bur or sand-blast, clean the ceramic surface with phosphoric acid, rinse and blow with oil-free compressed air.

Extraoral application:

According to the manufacturer's specifications

#### 3.1.2 Metal and amalgam

Roughen the surface with a diamond bur or sand-blast.

Then clean with a steam jet (extraoral) or acetone and dry with oil-free compressed air.

#### 3.1.3 Composite

Roughen the composite surface with a diamond bur or sand-blast, clean with phosphoric acid, rinse and dry with oil-free compressed air.

### 3.2 Application

Proceed as in point 1.3

### 3.3 Restorations

Use the restoration material according to the manufacturer's specifications.

## **4. Sealing dentine**

Proceed as in point 1.1, 1.2 and 1.3

## **Emergency measures, symptoms, antidotes**

If the product comes into contact with the oral mucosa, rinsing with water is sufficient. If the product comes into contact with eyes, rinse thoroughly with water. Consult an ophthalmologist for assessment and further treatment.

## **Note**

Product is only to be provided to dentists and dental technicians or on their behalf.

Keep out of the reach of children.

Only dispose of completely emptied packages.

## **Expiry date and labelling**

The expiry date and **LOT** code are found on the containers and external packaging.

Do not use the product beyond its expiry date.

## **Storage**

4-8 °C / 39-46 °F

Do not expose to direct sunlight or other heat sources.

## **Hygiene tips**

Observing the following instructions ensures optimal hygienic conditions for the use of ONE COAT 7 UNIVERSAL:

- Only use a disposable brush for one patient.
- Close the bottle with the sealing cap after use and disinfect with standard disinfectants.

## Caution

Federal law restricts this device to sale by or on the order of a dentist.

## Date of issue of instructions for use

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**Coltene/Whaledent AG**