Phosphoric Acid Etching Gel

TOKUYAMA EE-BOND

TOKUYAMA ETCHING GEL HV

REF AD02

Last updated 2019-10-31 Rev A

ENGLISH

Read all information, precautions and notes before using.

■ PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. TOKUYAMA ETCHING GEL HV contains 39 wt% phosphoric acid; the ideal viscosity of the gel ensures effective application. The strong color contrast allows the gel to be applied accurately. TOKUYAMA ETCHING GEL HV is rinsed away quickly, and the red color allows residue to be easily recognized after rinsing.

2. TOKUYAMA ETCHING GEL HV contains Phosphoric acid, Purified water, Thickerener, Colorant.

■ INDICATIONS

Acid etching enamel and dentin

■ CONTRAINDICATIONS

1. TOKUYAMA ETCHING GEL HV contains phosphoric acids. DO NOT use TOKUYAMA ETCHING GEL HV for patients allergic to or hypersensitive to acids.

2. DO NOT mix TOKUYAMA ETCHING GEL HV and sodium hypochlorite, mixing will generate Chloride gas which is harmful.

■ PRECAUTIONS

1) DO NOT use TOKUYAMA ETCHING GEL HV for any purpose other than those listed in these instructions. Use TOKUYAMA ETCHING GEL HV only as directed herein.

2) TOKUYAMA ETCHING GEL HV is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.

3) DO NOT use TOKUYAMA ETCHING GEL HV if the safety seals are broken or appear to have been tampered with.

4) If TOKUYAMA ETCHING GEL HV causes an allergic reaction or oversensitivity, discontinue its use immediately.

5) Use examination gloves (plastic, vinyl or latex) at all times when handling TOKUYAMA ETCHING GEL HV.

6) Avoid contact of TOKUYAMA ETCHING GEL HV with eyes, mucosal membrane, skin and clothing.

- If TOKUYAMA ETCHING GEL HV comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.

- If TOKUYAMA ETCHING GEL HV comes in contact with the mucosal membrane and skin, wipe the affected area immediately, and thoroughly flush with water and immediately contact a physician.

- If TOKUYAMA ETCHING GEL HV comes into contact with clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze, and flush with water.

- Instruct the patient to rinse his mouth immediately after treatment.

7) To avoid the unintentional ingestion of TOKUYAMA ETCHING GEL HV, do not leave it unsupervised within the reach of patients and children.

8) To avoid cross contamination, DO NOT reuse the disposable tip.

9) Close the syringe cap after use.

10) Near pulp exposures should be treated with a protective base prior to placing etching gel.

11) To avoid the mixing of TOKUYAMA ETCHING GEL HV and sodium hypochlorite (sodium hypochlorite included materials and medicaments) when using both in the same procedure, thoroughly rinse before using either.

12) Confirm the viscosity of TOKUYAMA ETCHING GEL HV when used for the first time and/or after storage for a long period of time.

■ STORAGE

1) Store at temperatures between 0 to 25°C (32 to 77°F).

2) DO NOT use after the indicated expiration date on the syringe.

■ DISPOSAL

Unused TOKUYAMA ETCHING GEL HV is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

■ CLINICAL PROCEDURES

Read and follow the instructions for etching procedures of the products being used when using TOKUYAMA ETCHING GEL HV.

1. Cleaning

If surface to be etched is not freshly cut, clean the surface using pumice, disk or a diamond bur.

2. Isolation

The rubber dam is the preferred method of isolation.

3. Etch
Attach the Disposable Tip after removing the cap of TOKUYAMA ETCHING GEL HV. Verify the flow of TOKUYAMA ETCHING GEL HV prior to applying intra-orally. Apply TOKUYAMA ETCHING GEL HV to the tooth structure to be etched.

4. Rinse

After appropriate time, rinse the etched surface thoroughly (at least 5 seconds) with water.

5. Dry

Dry all etched surfaces appropriately. DO NOT allow the etched surfaces to be contaminated by anything. If contamination occurs, re-etch, rinse and dry as above.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The manufacturer of the TOKUYAMA ETCHING GEL HV is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Product specifications of the TOKUYAMA ETCHING GEL HV are subject to change without notice. When the product specifications change, the instructions and precautions may change also.

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