

Bond-1™ SF
Solvent-Free SE Adhesive

**Solvent-free is
the way to be.**

PENTRON™

Why solvent-free?

Solvents such as acetone, alcohol, or water can be inadvertent contributors to weakened bond strengths.

Bond-1™ SF

Solvent-Free SE Adhesive



FEATURES & BENEFITS



SOLVENT-FREE

Solvents have been effectively removed while preserving the high bond strengths associated with conventional bonding agents.



NO AIR DRYING

Protects against technique-sensitivity and doesn't leave you guessing, "How dry is too dry?"



AN EVEN SPREAD

The unique handling properties enables an even spread of the material, ensuring complete coverage and optimal results.



SELF-ETCH

One-bottle solution—no need to acid etch, which saves time.



VERSATILE

So versatile, it can be used in conjunction with light cure and dual-cure materials.*



ONE COAT

Easy to use, reducing procedure time.

Bond-1™ SF Solvent-Free SE Adhesive helps eliminate a step and protect against technique-sensitivity issues.

Over drying can lead to common technique-sensitivity issues by leaving the dentin too dry and exposing the dentin tubules. Likewise, under drying can leave residual solvent in the resin, thus irritating nerves within the dentin tubules.

By eliminating the solvents from Bond-1 SF we have eliminated both the drying step and the need to worry about these common technique sensitive issues.

*Compatible with dual-cure materials only when the initial layer of dual-cure material is light cured.

Posterior restorations made easy.

1




Preparation of an MOD (3 surface) direct restoration.

2




Bond-1 SF is being applied with the flocked tip and rubbed for 20 seconds.

3

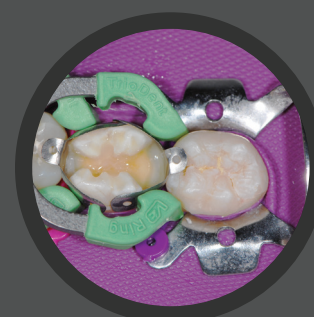


The unique handling properties allow for an even spread of the material, for a consistent bond every time.

4

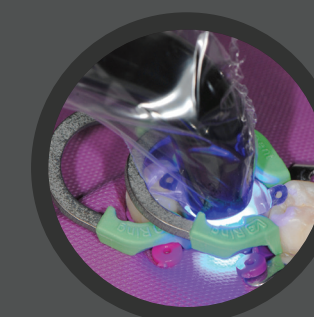

Light cure the bonding agent for 10 seconds.

5

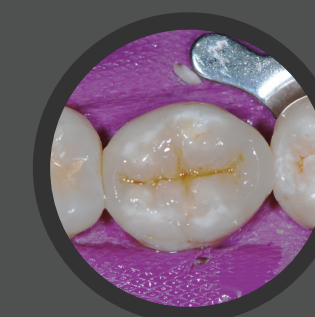

Final direct restoration accomplished with the Bond-1™ SF Solvent-Free SE Adhesive, Artiste™ Nano Composite.

Order today!

Syringes

The syringe offers a unique flocked needle tip for easy, direct placement onto the prep. Also includes a smear-free UV coated label for easy cleaning after each use.

N03N 2 - 1ml Syringes, 20-25 Gauge Flocked Needle Tips

Accessories

N03T 100 - 25 Gauge Flocked Needle Tips



Make the call!

Want Bond-1™ SF Adhesive?

Give your Pentron Sales Team a ring at:

855-748-2964

Or email:

PentronTeam@pentron.com

Have questions?

Chat with our Customer Service at:

800-551-0283

or visit:

www.pentron.com

PENTRON™

1717 West Collins Avenue

Orange, CA 92867

800.551.0283

855.748.2964

www.pentron.com