

Desident® CaviCide

Disinfectant/decontaminant cleaner for dental and medical devices: Bactericidal, Virucidal, Fungicidal, Tuberculocidal.

DIRECTIONS FOR USE:

FOR PRE-CLEANING DENTAL AND MEDICAL DEVICE SURFACES PRIOR TO DISINFECTION: Remove gross filth and heavy soil loads. Spray Desident® CaviCide directly onto dental and medical device surfaces. Allow it to remain visibly wet for 30 seconds. Wipe surface using a towel.

FOR PRE-CLEANING DENTAL AND MEDICAL DEVICES PRIOR TO DISINFECTION: Remove gross filth and heavy soil loads.

For use as a pre-cleaning de-contaminant spray: Place dental and medical devices in a container. Spray Desident® CaviCide so as to thoroughly drench all dental and medical device surfaces and allow it to remain visibly wet for 30 seconds. Remove and rinse dental and medical devices. Follow with appropriate cleaning and disinfection process.

For use as an ultrasonic pre-cleaning solution: Thoroughly rinse dental and medical devices under running water to remove visible gross debris. Please add 30 mL Desident® CaviCide per litre of water to ultrasonic unit. Immerse dental and medical devices in solution and activate ultrasonic unit for 5 minutes or longer, if necessary. Remove and rinse instruments. Follow with appropriate cleaning and disinfection process. Discard solution after each use.

For use as a manual pre-cleaner: Thoroughly rinse dental and medical devices under running water to remove visible gross debris. Please place dental and medical devices in a container with 30 mL Desident® CaviCide per litre of water. Scrub dental and medical devices while submerged for 30 seconds using a stiff bristle brush until visibly clean. Remove and rinse dental and medical devices. Follow with appropriate cleaning and disinfection process. Discard solution after each use.

FOR USE AS A DISINFECTANT ON MEDICAL DEVICE SURFACES: Spray Desident® CaviCide directly onto pre-cleaned dental and medical device surfaces, thoroughly wetting area to be disinfected. Allow dental and medical device surfaces to remain visibly wet for 3 minutes at room temperature (20 °C). Wipe surface using a towel or allow to air dry.

FOR USE AS A DISINFECTANT ON PRE-CLEANED NON-CRITICAL MEDICAL DEVICES¹, INSTRUMENTS AND IMPLEMENTS:

Medical devices must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow medical devices. Using either a tray or an ultrasonic unit, immerse medical devices in Desident® CaviCide for 3 minutes at room temperature (20 °C). Remove and rinse medical devices. Wipe dry prior to use. Discard solution after each use.

¹ Non-critical medical devices are items that come in contact only with intact skin.

Note: Critical and semi-critical devices must be followed by appropriate terminal sterilisation/high-level disinfection process.

Precautions:

This product maybe harmful to Humans and Domestic Animals. Avoid contamination of food. Avoid contact with eyes. May cause eye irritation.

First aid instructions:

Treatment for eye contact: Flush with water for 15 minutes. If irritation persists, seek medical attention.

Treatment for skin contact: Wash skin with soap and water.

Treatment for inhalation (breathing): Remove to fresh air. If irritation persists, seek medical attention.

Treatment for ingestion (swallowing): Do not induce vomiting. Give large amounts of water. Seek medical attention.

Storage and disposal:

Storage: Store in a cool place and keep container tightly closed. Avoid exposure to excessive heat.

Disposal considerations: Dispose of in accordance to local regulations.

FOR PROFESSIONAL USE ONLY

WARNING: In presence of ignition sources and excessive heat, this product may be flammable.

KEEP OUT OF REACH OF CHILDREN

Emergency Phone: +420 224 919 293

Toxicology Information Center

Na Bojišti 1, CZ-120 00 Praha 2

SpofaDental a.s.

a Kerr Company

Markova 238

CZ-506 46 Jičín

Tel: +420 493 583 111

Fax: +420 493 583 333

MADE IN CZECH REPUBLIC

Manufactured For:

KERR TotalCare

PROTECTING PEOPLE

Bactericidal, Virucidal, Fungicidal, Tuberculocidal

Desident® CaviCide

disinfectant/de-contaminant cleaner for dental and medical devices

200 mL, 700 mL, 5 litres

Item No. 4731221