<p><b>en - ENGLISH</b></p>
<p><span><b>Manufactured By: Pentron Clinical</b></span> <p>1717 West Collins Avenue Orange, CA 92867 USA 400-551-0283 / 714-516-7557 / www.pentron.com</p></p>
<p><b>Distribution Facility:</b> <p>Kerr Australia Pty. Ltd., Unit 6, 12 Mars Road Lane Cove West, NSW 2066 Australia +61-2-8870-3000</p></p>
<p><span><b>CE</b></span> <p>SpofaDental a.s. Markova 238, CZ-506 01 Jičín Phone: +420 493 583 111</p></p>

**Flow-It™ ALC** Flowable Composite

**DEVICE DESCRIPTION**  
Flow-It ALC (Accelerated Light Cure) is a micro-hybrid flowable composite with a flexural and tensile strength comparable to the best hybrid composites. Its excellent handling capabilities and just the right viscosity make it easy to use in an extensive range of applications.

**INDICATIONS FOR USE**  
Flow-It ALC is a flowable visible light curing hybrid composite restorative material indicated for all Class I, II, III, IV, V restorations

**CONTRAINDICATIONS**  
*None known*

- WARNINGS**
- Do not use this product on patients with a known allergy to or hypersensitivity to methacrylic and related monomers. Un cured methacrylate resin may cause contact dermatitis and harm the pulp. Avoid contact with the skin, eyes and soft tissue. Wash thoroughly with water and soap after contact. Get medical attention if the product comes into contact with the eyes.
  - Use examination gloves (plastic, vinyl, or latex) at all times to avoid the possibility of allergic reactions from methacrylic monomers.

- PRECAUTIONS**
- Use a rubber dam to isolate the oral cavity.
  - Avoid fluid contamination during placement.
  - Tips are for single-patient use only to prevent cross-contamination between patients.

<p><b>ADVERSE REACTION</b></p>
<p><i>None known</i></p>
<p><b>INSTRUCTIONS FOR USE</b></p>
<p><b>Preparation</b> Prepare the tooth as usual.</p>

**Ethc and Apply Bonding Adhesive**  
Use Bond-1™ One-Component Total Etch Bonding System or Bond 1™ SF Solvent-Free Self-Etch Adhesive or another adhesive system. Please refer to the manufacturer´s Instructions for Use for the specific bonding agent you are using.

**Place Flow-It ALC Flowable Composite**  
Apply Flow-It ALC directly with the bent needle tip provided in the kit. Be sure to cover the preparation thoroughly with Flow-It ALC. Do not build increments of more than 2mm thickness for proper light curing.\* Pull back on the syringe plunger after use to prevent material seepage.

**Light Curing**  
Halogen or LED (or light with output >1000 mW/cm<sup>2</sup>): Cure initial increments for 3 seconds, and the final increment for 6 seconds.  
With Halogen or LED (or light with output 600 – 1000 mW/cm<sup>2</sup>): Cure initial increments for 10 seconds, and the final increment for 20 seconds.  
**\*Note:** Due to their opacity, Universal

Opaque, White Opaque, Gingival Light and Gingival Dark shades should not be placed in increments of more than 1mm thickness to ensure effective polymerization.

**SHELF LIFE AND DISPOSAL**  
Refer to the outer packaging for the expiration date. Do not use after the expiration date. For proper disposal, always follow Federal or State laws.

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