# Icon Vestibular Etch

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English

### Product description

The etching gel Icon Vestibular Etch forms part of infiltration treatment with Icon. For the entire infiltration treatment, the following instructions for use must be strictly observed: Icon Vestibular

### Intended purpose

Removing the pseudointact surface layer in connection with infiltration treatment with Icon.

### Limitations of use

Sufficient drying and protection of the gingiva not possible.

### Indications

- Non-cavitated carious lesions
- Fluorosis
- Idiopathically or traumatically caused spots

### Contraindications

Do not use this material:  
 in cases of allergies to any of the ingredients or in cases of contact allergies.

## Patient target group

People treated in the course of a dental procedure.

## Intended users

Dentist

## Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must only be used on one patient for hygiene reasons.
- The etched surface must not be touched or contaminated with saliva or moisture until the treatment resumes. If contamination occurs after drying re-etch for approx. 10 seconds and repeat drying with Icon Dry.
- For effective treatment results a sufficiently dry working area is critical. Therefore, appropriate measures for a completely isolation must be taken. A liquid dam can be used as well. Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

## Recommended use (for image, see »Handling«)

Application of Icon Vestibular Etch:

- Screw the Vestibular Tip onto the Icon Vestibular Etch syringe.
- Apply an ample amount of Icon Vestibular Etch onto the lesion site by turning the syringe shaft carefully, let set for 2 minutes and activate it by moving (Fig.1). Remove excess material with dental wadding.
- Suck off Icon Vestibular Etch and rinse with water for at least 30 s. Then dry using oil-free and water-free air (Fig. 2).

## Interactions

Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

## Residual risks/side effects

- There are no known side effects to date. The residual risk of hypersensitivity in relation to allergies to components of the material cannot be ruled out.
- Contact of Icon Vestibular Etch with the oral mucosa causes a white surface coloration. This will subside after a few days.

## Warnings / precautions

- Keep out of the reach of children.
- Caution! Icon Vestibular Etch can cause chemical burns
- Avoid contact with soft oral tissue, eyes, and skin. In case of inadvertent contact rinse with copious amounts of water, and, if necessary, consult a physician.
- Limit the application to the surface to be treated. Avoid contact of the etching gel with the gingiva, exposed dentine, root element and adjacent teeth.
- Use only the enclosed syringe and application tip to apply the etching gel in the patient's mouth.
- Dispose of application tip after use. Reattach syringe closure.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

## Performance characteristics of the device

pH value  $\leq 0$

## Storage / disposal

- Store in a dry place at 2 to 25°C (36 to 77°F)!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

## Composition

Hydrochloric acid, pyrogenic silicic acid, surface-active substances

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