ENGLISH

Read all information, precautions and notes before using.

PRODUCT DESCRIPTION AND GENERAL INFORMATION

1) ESTELITE Σ QUICK is a light-cured, radiopaque, submicron filled composite resin for use in indirect posterior restorations and is indicated for all classmate restorations. ESTELITE Σ QUICK contains Bis-GMA and triethylene glycol dimethacrylate monomers and glass fiber fillers. A high filler load decreases polymerization shrinkage. All inorganic filler contained in ESTELITE Σ QUICK is a spherical submicron filler (mean particle size: 0.2 μm, particle size range: 0.1 - 0.3μm) that is designed to deliver excellent gloss retention and wear resistance. The monomer matrix contains Bis-GMA and triethylene glycol dimethacrylate.

2) ESTELITE Σ QUICK implements Radical-Amplified Photopolymerization initiator (RAP). Compared to conventional resin composite, ESTELITE Σ QUICK can be dispensed by one-third using a light curing unit with a wavelength range of 400 to 500 nm (peak: 470nm) compared to our conventional resin composite. Please see the table depicting the relationship between curing time and increment depth (**INDICATIONS FOR FILLING AND CURING**).

3) RAP technology facilitates a longer working time for ESTELITE Σ QUICK.

4) ESTELITE Σ QUICK is provided either in SYRINGE or Pre Loaded Tip(PLT).

SHADE

A1, A2, A3, A3.5, A4, A5, B1, B2, B3, B4, C1, C2, C3, O1, O2, O3, OPAZ, BW (Bleach White), WE (White Enamel) and CE (Clear Enamel).

* Opalescent shades (O1, O2, O3) have adequate opacity designed for blocking out the dark shine that comes through the oral cavity (Class III and Class IV restoration). The opalescent shades also serve as a dentin layer for the multi-shade layering technique. Shades O1, O2 and BW closely match those of primary teeth. However, they are not designed for masking metal or a darkly stained area such as tetraethylene stained.

- OPAZ is designed for masking slight stain or reconstructing a highly opaque tooth (use as a thin layer at the lingual wall of class III and IV).

- Please check with your local distributor for available shades.

INDICATIONS

- Direct anterior and posterior restorations including the occlusal surface and all carious classes

- Discoloration of composite veneer

- Repair of porcelain/composite

CONTRAINDICATIONS

ESTELITE Σ QUICK contains methacrylic monomers. DO NOT use ESTELITE Σ QUICK for patients allergic to or hypersensitive to methacrylic and related monomers.

PRECAUTIONS

1) DO NOT use ESTELITE Σ QUICK for any purpose other than those listed in these instructions. Use ESTELITE Σ QUICK only as directed herein.

2) ESTELITE Σ QUICK is designed for sale and use by licensed dental care professionals only.

If it is not designed for sale or it is not suitable for use by non-dental care professionals.

3) DO NOT use ESTELITE Σ QUICK if the safety seals are broken or appear to have been tampered with.

If ESTELITE Σ QUICK causes an allergic reaction or oversensitivity, discontinue its use immediately.

5) Use examination gloves (plastic, vinyl, or latex) at all times when handling ESTELITE Σ QUICK to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If ESTELITE Σ QUICK comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.

6) Avoid contact with eyes, mucous membrane, skin and clothing.

- If ESTELITE Σ QUICK comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.

- If ESTELITE Σ QUICK comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration.

- If ESTELITE Σ QUICK comes into contact with the skin or clothing, immediately saturate

- Rinse the patient to rinse his mouth immediately after treatment.

7) ESTELITE Σ QUICK should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.

8) To avoid the unintentional ingestion of ESTELITE Σ QUICK, do not leave it unsupervised within the reach of patients and children.

9) Clean the placement instruments with alcohol after use.

10) When using a light-curing unit, protective eye shields, glasses or goggles should be worn.

INDICATIONS FOR FILLING AND CURING

Be sure to light-cure ESTELITE Σ QUICK extraorally and check the time needed for complete hardening of ESTELITE Σ QUICK with your light curing unit before performing the bonding procedure. The following table illustrates the relationship between curing time and increment depth.

Relationship between curing time and increment depth:

Light type	Intensity (mW/cm ²)	Curing time (seconds)	Increment depth (mm) ⁽¹⁾					
			A1, A2, A3, A3.5, B1, B2, B3, C1, C2, O1, O2, CE(Inc.), BW, WE	A4, A5, B4, C3, O3, OPAZ	B1, B2, B3, C1, C2, O1, O2, CE(Inc.), BW, WE	A1, A2, A3.5, B1, B2, C1, C2, O1, O2, CE (Inc.), BW, WE	A4, A5, B4, C3, O3, OPAZ	
Halogen	400	10	1.9	1.6				
		20	2.2	1.9				
	600	10	2.0	1.7				
		20	2.5	2.0				
LED	800	5	1.8	1.6				
		10	2.2	1.9				
	260	10	1.7	1.4				
		20	2.0	1.7				
Plasma Arc	900	5	1.7	1.5				
		10	2.0	1.8				
	950	3	1.9	1.5				

⁽¹⁾ Increment depth was determined on the basis of test results performed in accordance with section 7.10 "depth of cure" of ISO4049.

SPECIAL NOTES FOR THE USE OF PLT

1) PLTs are designed for single patient use only. To avoid cross infection, Do not re-cap and/or re-use the PLT tip once material has been dispensed for that patient.

2) Dispensers are not provided for ESTELITE Σ QUICK. Use a dispenser that fits the PLT of ESTELITE Σ QUICK. For proper use and proper disinfection, see manufacturer's instructions.

3) Apply the composite at normal room temperature (18 - 30 °C / 62 - 84 °F). Material that is very cold might be difficult to extend.

4) Use light, controlled pressure to prevent any continued extrusion of material following pressure release.

STORAGE

1) Store ESTELITE Σ QUICK at a temperature between 0 - 25 °C (32 - 77 °F).

2) AVOID direct exposure to light and heat.

3) DO NOT use ESTELITE Σ QUICK after the indicated date of expiration on the syringe or PLT package.

DISPOSAL

To safely dispose of excess ESTELITE Σ QUICK, extrude unused portion from SYRINGE or PLT and light-cure before disposal.

CLINICAL PROCEDURE**1. Cleaning**

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

Select the appropriate shade of ESTELITE Σ QUICK prior to placing the rubber dam using the shade guide provided for ESTELITE Σ QUICK.

- Complete the shade selection within 5 minutes; desiccated teeth are lighter than wet or moist teeth.

- Lightness (color value) is primary importance on shade selection.

- When wavering in shade selection, select the lighter (high value) shade.

- In case of whitened teeth select the shade a few weeks after the completion of whitening. Whitened teeth may become slightly darker.

3. Isolation

A rubber dam is the preferred method to isolate the tooth from contamination.

4. Cavity Preparation

Prepare the cavity and rinse with water. Add bevels to the enamel margins of anterior preparations (class III, IV, V), as well as to the margins of posterior preparations (class III, IV, V) to prevent any discrepancies between the cavity margins and restoration, thereby enhancing both esthetics and retention.

- Scalloped (undulant) bevel could be desirable depending on the esthetics and retention.

- In case of porcelain/composite repairs, roughen the surface with a bur and a diamond point to prepare the area for adhesion; apply phosphoric acid etch for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.

5. Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT use EUGENOL BASED MATERIALS to protect the pulp

as these materials will inhibit curing of bonding agent and/or ESTELITE Σ QUICK.

6. Bonding system
Apply a bonding system according to manufacturer's instructions.

7. Dispensing

Please read **■ SPECIAL NOTES FOR THE USE OF PLT before use.**
- Load the PLT into a dispenser that fits the PLT.

- Remove the PLT cap.
- Extrude the paste directly into cavity or indirectly on the mixing pad.

7-2. SYRINGE
Remove the syringe cap. Extrude the paste onto a mixing pad by turning the handle clockwise. Re-cap the syringe immediately after dispensing.

7-3. PLT
DO NOT apply unnecessary force to the syringe immediately after removing from the refrigerator.

8. Filling and contouring
Fill the cavity incrementally. Increments should not exceed the indicated depth (please refer to the aforementioned table).

- DO NOT mix ESTELITE Σ QUICK with other brands of resin composite or other shades of ESTELITE Σ QUICK to avoid incomplete cure or voids.

9. Curing
Light-ease each increment for at least the indicated time (please see aforementioned table), keeping the curing light tip within a distance of 2 mm from the increment.

- If other brands of composite resins are layered over the cured composite, follow their instructions.

10. Finishing
Sand and polish the restoration. For finishing, use fine finishing diamond points. To erase the demarcation between the composite and the natural tooth rotate a 12 fluted carbide bur slowly along the margins without water. Use fine finishing metal strips or vinyl strips for proximal surfaces. For polishing, polish with rubber points or any suitable polishing tool, such as PoGo™ (DENTSPLY Caulk), Sof-Lex™ (3M ESPE), Identoflex™ HiLuster Dia Polishers (Kerr), or D-FINE™ Hybrid Diamond (COSMEDENT, Inc.).

* PoGo, Sof-Lex, Identoflex and D-FINE are registered trademarks of Tokuyama Dental Corporation.

- Inspect the patient adequately on the maintenance of the restoration after treatment.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer of ESTELITE Σ QUICK is not responsible for damage or injury caused by misuse of this product. It is the responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Product specifications of ESTELITE Σ QUICK are subject to change without notification. When the product specification, change the instruction and precautions may change also.

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