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PORCELAIN ETCHANT (10% HYDROFLUORIDRIC ACID)

ENGLISH

DEFINITION

10% hydrofluoridric acid.

INDICATIONS

- Cementation of all-ceramic (porcelain) restorations.
- Repair of porcelain with composite resin.
- Etching of orthodontic brackets to be cemented in porcelain.

COMPOSITION

Hydrofluoridric acid, water, thickening agent and pigment.

TECHNIQUE FOR USE

A. CEMENTATION OF ALL-PORCELAIN RESTORATIONS

1. Isolate the external surface of the porcelain restoration with a thin layer of wax.

2. Etch the internal surface with ANGELUS® PORCELAIN ETCHANT (10% HYDROFLUORIDRIC ACID) for 1 minute*. Wash abundantly with water and dry thoroughly.

* *Leucite-based porcelains are etched only for 20 (twenty) seconds.*

3. Apply SILANE ANGELUS® on the etched surface for 1 minute and gently dry with air.
4. Proceed with cementation using a resin cement, according to the manufacturer's instructions of the cement.

B. PORCELAIN REPAIR

1. Make a 2 to 3 mm bevel at the fractured area and wash with an air/water spray.
2. Etch the beveled area with ANGELUS® PORCELAIN ETCHANT (10% HYDROFLUORIDRIC ACID) for 1 minute. Wash abundantly with water and dry thoroughly.
3. Apply SILANE ANGELUS® for 1 minute and gently dry with air.
4. Apply bonding agent FUSION-DURALINK® ANGELUS® and a restorative composite resin, according to the manufacturer's instructions.

PRECAUTIONS

- Hydrofluoridric acid is corrosive. Contact with enamel, dentin, eyes, skin and oral tissues must be avoided;
- During use, it is recommended that eye protection, gloves, and mask be used by the dentist and eye protection be used by the patient;
- When applied in the patient's mouth, oral tissues

should be protected with vaseline and, whenever possible, use rubber dam isolation;

- In case of accidental contact with eyes, skin or oral tissues, wash abundantly with water. If necessary, seek medical attention;
- Keep the package closed after use;
- Do not use on patients with known sensitivity to any of the components.

STORAGE

Store the product in a dry place away from light.

PROFESSIONAL USE

VITRO-CERAM® is a registered trademark of ANGELUS®. In-Ceram is a registered trademark of Vita. Procera is a registered trademark of Nobel Biocare.

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