Master all your challenges

*initial*TM
Lustre Pastes NF
V-shades

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GC Initial IQ Lustre Pastes NF V-Shades

EN Intended Use

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EN Colour deepness and lifelike translucency

GC Initial IQ Lustre Pastes NF (LP NF) are 3-dimensional ceramic stains developed to bring colour deepness and lifelike translucency for the complete GC Initial ceramic range. Thanks to their "New Formula" you can now really bring your crowns and bridges to life with this dedicated Lustre Pastes NF Set, with an optimal match to both "low" and "high" CTE ceramics. Available in a ready to use consistency, the Lustre Pastes NF are based on fine ceramic particles and can be applied in a thicker layer where they will exhibit unsurpassed vitality and a very natural glaze to your restorations.

Features

- Just one dedicated kit for all different indications for the complete GC Initial ceramic range.
- The GC Initial IQ Lustre Pastes NF are based on fine ceramic particles, enabling a thicker application compared to conventional stains.
- Improved and unique paste medium, it has a fine thixotropic property for easy and precise applications.
- Available in a ready to use consistency (1 x Neutral, 1 x Value, 4 x Body Shades, 10 x Enamel Effect Shades).
- The high fluorescence of the Lustre Pastes ensure the look of natural teeth.

Indications

The Lustre Pastes NF are a one for all solution for the complete Initial line from GC. It's possible to use the Lustre Pastes NF in a wide CTE – Range between 6,9 and 13,3 in order to work with nearly all types of ceramic. These Lustre Pastes NF represent a modern alternative for conventional staining and glazing, as well as for the colour, brightness, grey value and surface texture corrections of all of your ceramic crowns and bridges. Due to the unique properties it's just a question of minutes to individualize the occlusal surface of your crown and bridge work, to paint on white bands, decalcifications, hair lines and crack lines or to characterize artificial porcelain teeth.

Keep in mind: What you paint (on) is what you get (after firing)

GC Initial IQ Lustre Pastes NF Set

Contents:

1x Neutral, 1 x Value, 4 x Body Shades, 8 x Enamel effect shades, Diluting Liquid, Refresh Liquid, 1 mixing dish, 2 brushes, Picture IFU

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EN Note: Mix the material well before use.

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EN Lustre Paste "Diluting Liquid":

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.

Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, never dilute directly in the jars.

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EN Lustre Paste "Refresh Liquid":

The 'Refreshing Liquid' is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties. The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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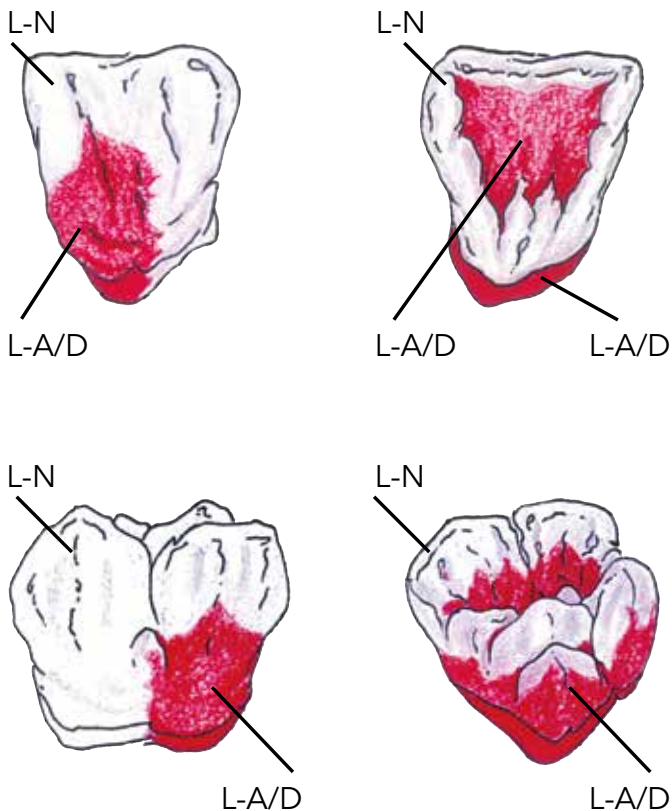
EN Important Information: Please always stir up material well in the jars before use. Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After prepolishing with a ceramic polisher the finished crowns & bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure. The Lustre Paste Neutral is coated on the entire surface of the sandblasted restoration. This coating is clearly thicker than what we know from a normal glaze firing. Note: These Lustre Pastes may not come in contact with water. The requested V-Classic Shade is created by a more or less intensive application of the Lustre Pastes "Body A - D". This process, which only takes 2 minutes, is everything which is necessary to create V-Classic shades.

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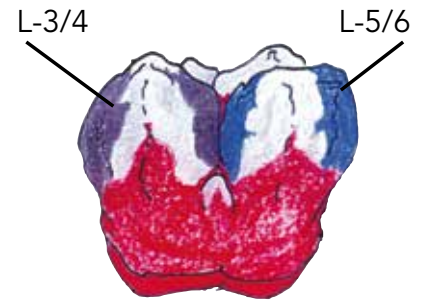
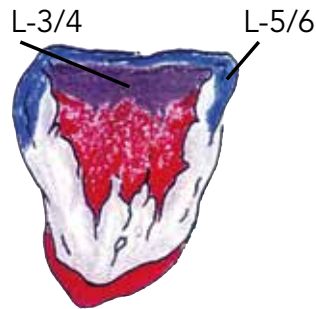
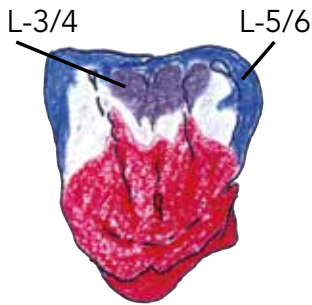
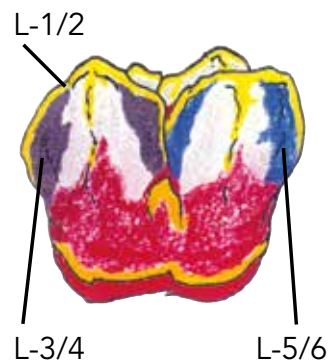
EN Lustre Pastes NF Application - Basic Step by Step Instruction

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EN To build up a high aesthetic solution for the patient use our advanced step by step instruction:
In order to achieve the higher grey value of the incisal area with darker teeth (A3.5; A4; C3; C4), 2 grey Effect Shades can be used: L-3 (Light Grey) and L-4 (Dark Grey). For eventual further individualisation, you can use the whitish Lustre Pastes NF: L-1 (Vanilla) and L-2 (White) and L-V (Value) for brightening the restorations. 2 bluish Effect Lustre Pastes NF: L-5 (Light Blue) and L-6 (Dark Blue) for a bluish opalescent appearance in the incisal area. Greenish Effect Lustre Paste NF: L-8 (Olive) for a greenish effect. Can be mixed with all Lustre Pastes NF.

In order to achieve an even higher „Transparency“ the 2 greyish Effect Lustre Pastes NF L-3/L-4 can be used as well. All GC Initial Spectrum Stains / GC Initial MC INvivo Stains can be used to achieve even more possibilities of individualization.

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EN The coloured, overburned dentin body is first fully covered with CL-F (MC or Zr-FS) and then layered with Opal/Incisal masses on the final shape of the tooth. (See also Initial Manual)

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EN The finished with the Initial Lustre Pastes NF glaze fired crowns are indistinguishable from high end, polychrome layered restorations.

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DE FAQ

FR FAQ

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EN **Basic Rule: What you paint (on) is what you get (after firing)**

Q1. How can I dilute (adapt the consistency of) the Lustre Pastes NF?

Use the Lustre Pastes NF Diluting Liquid to adapt the consistency. Do NOT use water to dilute. LP may not come in contact with water (see page 9).

Q2. What can I do when the consistency of the Lustre Pastes NF is too dry?

Use the Refresh Liquid to recover the Lustre Pastes NF in case of dryness (see page 9).

Q3. How do we change the texture of the Lustre Pastes NF?

The requested surface texture/smoothness of the Lustre Pastes NF layer can be changed by soft vibration or condensing the applied pastes.

Q4. I do not have a shiny appearance after firing the Lustre Pastes NF, what is the cause?

Check the firing temperatures, increase the final temperature. When the Lustre Pastes NF are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

Q5. Can I apply more than one layer of Lustre Pastes NF?

Yes, you can do further firings, keeping the same firing temperatures.

Q6. Can I mix the Lustre Pastes NF with other Initial powders?

You can use the GC Initial Stain powders with moderation to intensify the Lustre Pastes NF. Mix the required amount of stain powder with the LP NF Diluting Liquid and mix it with the required LP NF or directly mix the stain powders with f.i. the Lustre Paste Neutral (L-N).

Q7. I have finished my veneered crown (MC/LF) and I want to use the Lustre Pastes NF without loosening the surface texture of the prepared crown.

You can easily dilute the LP to have the optimal consistency in order not to loose the surface texture and still have a nicely coloured and shiny crown, just apply in a thin layer.

Q8. Are the Lustre Pastes NF sufficient abrasion resistant?

Yes, the LP NF are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering colour deepness and lifelike translucency.

Q9. I have to change the colour, brightness or the grey value of a crown made with the layering technique. What can I do?

Use the Lustre Pastes NF as shown on page 16 to modify your crown and bridge work.

Q10. After firing the colour of my work is spotted?

The Body Colour needs to be massaged into the Lustre Paste Neutral (L-N) on your crown or should be mixed equally on an external mixing board in advance. To disperse the colour pigments it is necessary to condensate the Lustre Pastes NF before the glaze firing by soft vibration.

Q11. The colour doesn't fit with the V-Classic Shade guide?

Different material thicknesses cause different colour effects. Therefore, it may be necessary in individual cases, to mix an A-Body Colour with a B-Body or even with L8 (Olive). A visual check is always possible.

Q12. The glaze firing is bleakly, spotted or cracked?

The Lustre Pastes NF came partly or completely in contact with water.

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EN Medical Device Regulation Requirement

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

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Handwriting practice lines consisting of multiple horizontal dotted lines for tracing and writing.

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 EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240
Interleuvenlaan 33, B-3001 Leuven
Tel. +32.16.74.10.00, Fax. +32.16.40.02.14
info.gce@gc.dental, www.gceurope.com

RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 W. 127th Street, Alsip, IL 60803 U.S.A.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
Tel: +1-708-597-0900
www.gcamerica.com

GC Germany GmbH
Seifgrundstr. 2, D-61348 Bad Homburg
Tel. +49.61.72.99.59.60, Fax. +49.61.72.99.59.66.6
info.germany@gc.dental, www.germany.gceurope.com

GC ITALIA S.r.l.
Via Calabria 1, I-20098 San Giuliano Milanese
Tel. +39.02.98.28.20.68, Fax. +39.02.98.28.21.00
info.italy@gc.dental, www.italy.gceurope.com

GC UNITED KINGDOM Ltd.
Coopers Court, Coopers Court, Newport Pagnell, UK-Bucks. MK16 8JS
Tel. +44.1908.218.999, Fax. +44.1908.218.900
info.uk@gc.dental, www.uk.gceurope.com

GC FRANCE s.a.s.
8 rue Benjamin Franklin, F-94370 Sucy en Brie Cedex
Tel. +33.1.49.80.37.91, Fax. +33.1.45.76.32.68
info.france@gc.dental, www.france.gceurope.com

GC IBÉRICA - Dental Products, S.L.
Edificio Codesa 2
Playa de las Américas, 2, 1º, Of. 4, ES-28290 Las Rozas, Madrid
Tel. +34.916.364.340, Fax. +34.916.364.341
comercial.spain@gc.dental, www.spain.gceurope.com

GC AUSTRIA GmbH
Tallak 124, A-8103 Rein bei Graz
Tel. +43.312.45.40.20, Fax. +43.312.45.40.20.40
info.austria@gc.dental, www.austria.gceurope.com

GC Europe NV - Benelux Sales Department
Researchpark Haasrode-Leuven 1240
Interleuvenlaan 33, B-3001 Leuven
Tel. +32.16 74.18.60
info.benelux@gc.dental, http://benelux.gceurope.com

GC EUROPE N.V. - East European Office
Siget 19b, HR-10020 Zagreb
Tel. +385.1.46.78.474, Fax. +385.1.46.78.473
info.eeo@gc.dental, www.eeo.gceurope.com

GC NORDIC AB - Finnish Branch
Bertel Jungin aukio 5 (6. kerros), FIN-02600 Espoo
Tel. +358 40 9000 757
info.finland@gc.dental, www.finland.gceurope.com

GC NORDIC AB - Danish Branch
Scandinavian Trade Building
Gydevang 39-41, DK-3450 Allerød
Tel: +46 768 544 350
info.denmark@gc.dental, http://nordic.gceurope.com

GC NORDIC AB
c/o Lundin Revisionbyrå
Erik Dahlbergsgatan 11B - S-411 26 Göteborg
info.nordic@gc.dental, www.nordic.gceurope.com

GC AUSTRIA GmbH - Swiss Office
Bergstrasse 31, CH-8890 Flums
Tel. +41.81.734.02.70, Fax. +41.81.734.02.71
info.switzerland@gc.dental, www.switzerland.gceurope.com

GC AUSTRALASIA DENTAL PTY LTD
1753 Botany Rd, Banksmeadow NSW 2019, Australia
Tel: +61-2-9301 8200, Fax: +61-2-9316 4196

GC SOUTH AMERICA
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - Tel: +55-11-2925-0965 - CNPJ: 08.279.999/0001-61
RESP. TÉC: Mayara de Santis Ribeiro - CRO/SP 105.982

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541
Tel: +65 6546 7588