

Simile™ and Artiste™
Nano Universal Composites

**A little nano goes
a long way.**

PENTRON™

Simile™ Nano

Why Simile?

Simile Nano Universal Composite

- You prefer a firm viscosity (like soft dough).
- You appreciate a simple technique and high shade selection to find your match.



Simile™ Nano

SIMILE NANO

FEATURES & BENEFITS



SCULPTABLE

A firmer viscosity with soft handling designed to not slump or stick to the instrument for easy placement.



EXTRAORDINARY SHINE

Visible as soon as you handle it.




POLISHABILITY

Nano-hybrid fillers produce an easy, smooth polish that lasts over time.



RADIOPAQUE

Easy for the clinician to see on an x-ray.

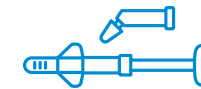

CONVENIENT DELIVERY SYSTEM

Choice of single dose or syringe.



LOW WEAR RATES

For beautiful long lasting restorations.



BODY SHADES

- A0, A1, A2, A3, A3.5, A4
- B0, B1, B2, B3, B4
- C1, C2, C3, C4
- D2, D3, D4
- Opaque Universal, Clear Incisal

Color swatches not accurately representative of actual color.



INDEPENDENTLY PROVEN:



“Nanosized filler particles provide an excellent final polish.”

“Very good interproximal contacts are achieved.”

Simile Nano and Artiste Nano perform well on any tooth classification or location.



CLASS 1



CLASS 2



CLASS 3



CLASS 4



CLASS 5

Artiste™ Nano

Why Artiste?

Artiste Nano Universal Composite

- You prefer a creamy viscosity (like peanut butter).
- You're an 'Artiste' and want exceptional polishability and performance.



Artiste™ Nano

ARTISTE NANO

FEATURES & BENEFITS



IDEAL HANDLING

Non-slumping, non-sticky, creamy consistency for effortless sculpting and shaping.



HIGHLY ESTHETIC

Exceptional polishability and performance.




RADIOPAQUE

Easy for the clinician to see on an x-ray.



INTUITIVE SHADING SYSTEM

Follows the VITA® shade guide for easy reference. Includes a bleach & an opaque shade to match the patient's dentition closely.

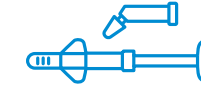

CONVENIENT DELIVERY SYSTEM

Choice of single dose or syringe.



BODY SHADES

- A1, A2, A3, A3.5
- B1, B2, B3
- C1, C2, C3
- Bleach, A Opaque

Color swatches not accurately representative of actual color.



INDEPENDENTLY PROVEN:



Simile Nano and Artiste Nano perform well on any tooth classification or location.

The Simile Nano Anatomical Layering Technique

Photos courtesy of Dr. Tarun Agarwal



Why Nano-Technology Matters

Pentron's Nano-engineered chemistry provides the perfect combination of esthetics and strength for easy polishability and performance. The high nano-particle formulation increases the total surface area of the filler content creating a stronger and more seamless integration with the resin matrix.

Physical Properties & Indications

FLEXURAL STRENGTH

SIMILE: 135 MPa
ARTISTE: 123 MPa

FLEXURAL MODULUS

SIMILE: 10.5 GPa
ARTISTE: 10.2 GPa

COMPRESSIVE STRENGTH

SIMILE: 287 MPa
ARTISTE: 353 MPa

POLISHABILITY

SIMILE: 59.5 gloss units
ARTISTE: 56.1 gloss units

RADIOACITY

SIMILE: 300% (or 3mm AI)
ARTISTE: 300% (or 3mm AI)

FILLER PARTICLE RANGE

SIMILE: 0.02-0.7 µm
ARTISTE: 0.02-0.7 µm

FILLER RATE (WT./VOL.)

SIMILE: 75%
ARTISTE: 75%

CLASS INDICATIONS

SIMILE: I, II, IV, V, VI
ARTISTE: I, II, IV, V, VI

The Artiste Nano Anatomical Layering Technique

Photos courtesy of Dr. Jeff Blank, DMD, PA



This surgical technique is the recommendation of Drs. Agarwal and Blank. Pentron is a medical device manufacturer and does not dispense medical advice. Clinicians should use their own judgment in treating their patients.

Order today!

Simile 4-Shade Syringe Trial Kit

N56S 4 - 4gm Syringes Shades: A2, A3, A3.5, Opaque Medium

Simile 10-Shade Syringe Kit

N56A 10 - 4gm Syringes Shades: A1, A2, A3, A3.5, B1, B2, B3, C2, Opaque Universal, Clear Incisal

Simile 10-Shade Single Dose Kit

N56C 100 - 0.25gm Single Doses Shades: A1, A2, A3, A3.5, B1, B2, B3, C2, Opaque Universal, Clear Incisal

Simile Syringe Refills

N56AT 1 - 4gm Syringe Shade A0
N56AA 1 - 4gm Syringe Shade A1
N56AB 1 - 4gm Syringe Shade A2
N56AC 1 - 4gm Syringe Shade A3
N56AD 1 - 4gm Syringe Shade A3.5
N56AN 1 - 4gm Syringe Shade A4
N56AU 1 - 4gm Syringe Shade B0
N56AE 1 - 4gm Syringe Shade B1
N56AF 1 - 4gm Syringe Shade B2
N56AG 1 - 4gm Syringe Shade B3
N56AO 1 - 4gm Syringe Shade B4
N56AL 1 - 4gm Syringe Shade C1
N56AH 1 - 4gm Syringe Shade C2
N56AK 1 - 4gm Syringe Shade C3
N56AP 1 - 4gm Syringe Shade C4
N56AJ 1 - 4gm Syringe Shade Incisal
N56AQ 1 - 4gm Syringe Shade D2
N56AR 1 - 4gm Syringe Shade D3
N56AS 1 - 4gm Syringe Shade D4
N56AI 1 - 4gm Syringe Shade Universal Opaque

Simile Single Dose Refills

N56BS 15 - .25gm Single Dose Shade A0
N56BA 15 - .25gm Single Dose Shade A1
N56BB 15 - .25gm Single Dose Shade A2
N56BC 15 - .25gm Single Dose Shade A3
N56BD 15 - .25gm Single Dose Shade A3.5
N56BM 15 - .25gm Single Dose Shade A4
N56BT 15 - .25gm Single Dose Shade B0
N56BE 15 - .25gm Single Dose Shade B1
N56BF 15 - .25gm Single Dose Shade B2
N56BG 15 - .25gm Single Dose Shade B3
N56BN 15 - .25gm Single Dose Shade B4
N56BH 15 - .25gm Single Dose Shade C1
N56BJ 15 - .25gm Single Dose Shade C3
N56BI 15 - .25gm Single Dose Shade C2
N56BO 15 - .25gm Single Dose Shade C4
N56BL 15 - .25gm Single Dose Shade Clear Incisal
N56BP 15 - .25gm Single Dose Shade D2
N56BQ 15 - .25gm Single Dose Shade D3
N56BR 15 - .25gm Single Dose Shade D4
N56BK 15 - .25gm Single Dose Shade Universal Opaque

Accessories for Simile and Artiste

N75A Dispenser Gun - Single Dose

Artiste Syringe Refills

N36AA 1 - 4gm syringe shade A1 Dentin/Body
N36AB 1 - 4gm syringe shade A2 Dentin/Body
N36AC 1 - 4gm syringe shade A3 Dentin/Body
N36AD 1 - 4gm syringe shade A3.5 Dentin/Body
N36AF 1 - 4gm syringe shade B1 Dentin/Body
N36AG 1 - 4gm syringe shade B2 Dentin/Body
N36AH 1 - 4gm syringe shade B3 Dentin/Body
N36AJ 1 - 4gm syringe shade C1 Dentin/Body
N36AK 1 - 4gm syringe shade C2 Dentin/Body
N36AL 1 - 4gm syringe shade C3 Dentin/Body
N36AQ 1 - 4gm syringe shade Bleach Dentin/Body
N36AOA 1 - 4gm syringe shade A Opaque

Artiste Single Dose Refills

N36BA 10 - .25gm Single Doses Shade A1 Dentin/Body
N36BB 10 - .25gm Single Doses Shade A2 Dentin/Body
N36BC 10 - .25gm Single Doses Shade A3 Dentin/Body
N36BD 10 - .25gm Single Doses Shade A3.5 Dentin/Body
N36BF 10 - .25gm Single Doses Shade B1 Dentin/Body
N36BG 10 - .25gm Single Doses Shade B2 Dentin/Body
N36BH 10 - .25gm Single Doses Shade B3 Dentin/Body
N36BJ 10 - .25gm Single Doses Shade C1 Dentin/Body
N36BK 10 - .25gm Single Doses Shade C2 Dentin/Body
N36BL 10 - .25gm Single Doses Shade C3 Dentin/Body
N36BQ 10 - .25gm Single Doses Shade Bleach Dentin/Body
N36BOA 10 - .25gm Single Doses Shade A Opaque

Make the call!

Want Simile Nano or Artiste Nano Composite?

Give your Pentron Sales Team a ring at:

855-748-2964

Or email:

PentronTeam@pentron.com

Have questions?

Chat with our Customer Service at:

800-551-0283

or visit:

www.pentron.com

Data* on file. The trademarks noted are owned by their respective owners, registered in the U.S. and other countries.
Copyright © 2017 Pentron Clinical

PENTRON™

1717 West Collins Avenue

Orange, CA 92867

800.551.0283

855.748.2964

www.pentron.com