EN

Super System Retreat One 25/06 (SS RO) Step-by-Step Procedure

Retreat One File is a single file for the removal of root canal filling materials combined with 3D activator files

Recommended Speed and Torque Values (300- 600 rpm, 2.5 Ncm)

Here is the step-by-step procedure for using the Super System Retreat One (SS RO) to remove root canal filling material in retreatment cases:

1. The Retreat One file works best when the canal is dry, so it's important to ensure dryness during working.
2. It is suggested to use small-sized round burs to remove root canal filling remnants in the canal orifice before the Retreat One file.
3. Coronal Third: Use Retreat One file with a speed of 500 -600 rpm in the coronal third for removal of canal filling material. Clean any debris and chips on the file flutes after each removal of the canal from the canal. Then, irrigate the canal abundantly with sodium hypochlorite and dry it.
4. Mid Third: Remove the canal filling in the middle third at a speed of 400 rpm with dry canals.
5. Apical Third: After irrigating and drying the canals again and cleaning the files, remove the apical canal filling at a speed of 300- 350 rpm to reduce the risk of pushing canal filling beyond the apex and post-procedure pain.

Additionally, if there is an existing ledge formation in the previous root canals this area can be felt thanks to the low speed and deepening of the ledge is prevented.

If ledge formation is felt in the previous root canals, the low speed can help prevent the deepening of the ledge

If the previous canal filling was short, use the Retreat One file at the length where the old canal filling ended. To reach the apical point beyond this length, use the 10/06 file in the SS MI kit or the 10/04 file in the SS AK.

After completing the canal enlargement, clean the canals at 800 rpm with the "Activator File" in a dry canal, followed by the same process with irrigation solution to increase efficiency. The activator file can be used after each step of the Retreat One file to enhance the removal of the canal filling.

Recommended speed settings according to the root canal region are:

- Coronal Third: 400 – 600 rpm - Mid Third: 300 - 400 rpm - Apical third: 300-350 rpm

For larger canals, complete the procedure with SS 35/04 and 45/02 larger files

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