ZONEfree (REF. 27041) / ZONE (REF. 27047) Automix Syringe Temporary Cement / Provisorium Zement / Ciment Provisoire

Method of use:

1. Remove the cap from the ZONEfree / ZONE automix syringe with a 90° turn counter clockwise.
2. Place and lock a mixing tip on the syringe with a 90° turn clockwise.
3. Apply a thin layer of ZONEfree / ZONE inside the temporary crown or bridge. The cement is automatically mixed in the mixing tip.
4. Seat crown or bridge firmly in the mouth.
5. At room temperature (22°C), ZONEfree Translucent Temporary Cement has a setting time of 2 min. and ZONE Temporary Cement has a setting time of 2.5 min.
6. Remove any excess cement after 2 min.

Where longer setting time or less strength is desired, add some Vaseline (petrolatum) or mineral oil to the cement.

Important: Do not leave ZONEfree / ZONE Temporary Cement in the mouth for a period longer than 30 days from placement. Store at room temperature.

This hard-setting, crystalline cement assures consistent superior retention, yet permits easy removal and clean up. ZONEfree / ZONE Temporary Cement is compatible with all provisional materials, will not affect the temporary crown, is kind to the pulp and tissue and will not decalcify dentin. Zinc-oxide; non-eugenol.

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