

myTines

small, medium, large
and myTines GO

extremities for
myClip 2.0 &
myRing Forte

Advantages

Replaceable,
interchangeable
and autoclavable
plastic extremities

Increased stability
of the rings in wide
and extreme wide cavities

V-shaped for
better positioning
over the wedge



polydentia
swiss manufacture 

sectional matrix systems
accessories

polydentia.ch





myTines

small, medium, large
and myTines GO

new!
myTines GO

in cooperation with **STYLE ITALIANO™**

Testimonial



"The new bulk restorative materials represent an extraordinary evolution in the industry.

Today, we can achieve restorations that once seemed unthinkable, thanks to the extraordinary chemical and physical properties of these innovative materials. An ideal partner for these new strategies is the recent Polydentia QuickmatFIT matrices, which enable a more natural restoration profile.

In addition, the myTines GO ring's extremities offer the possibility of applying the matrix in medium- and large-sized cavities in a completely new way, overcoming the limitations of traditional systems. All this results in larger restorations in less time."


Dr. Giuseppe Chiodera,
Italy



Images courtesy of Dr. Giuseppe Chiodera



for more
information

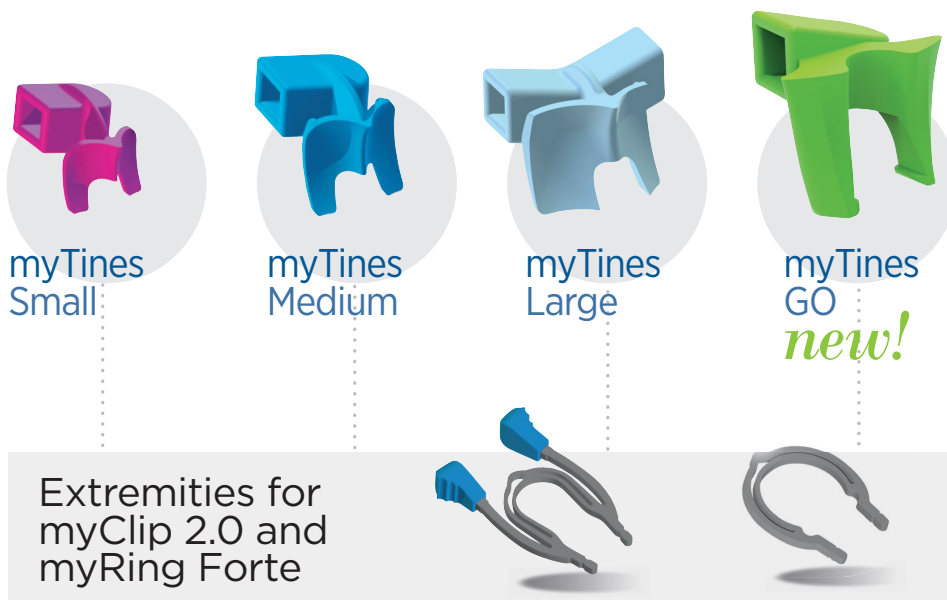

Which myTines do I use?




LOW crown
HEIGHT teeth

Use myTines
Small



NORMAL
crown
HEIGHT teeth

Use myTines
Medium

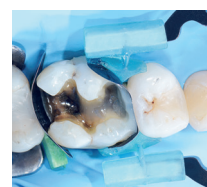

WIDE CAVITY
preparations/
MISSING cusp

Use myTines
Large



WIDE & EXTREME
WIDE CAVITIES
MISALIGNED
teeth*
MISSING cusps

Use myTines
GO

* wedge in place labial to palatal

myTines are versatile extremities for myRing Forte & myClip 2.0, developed for a precise adaptation of Polydentia's sectional matrices to a wide range of clinical situations and teeth with different crown heights.

In case of dissimilar tooth morphologies, the ability to combine different myTines (Small, Medium, Large and myTines GO) on the same Polydentia's sectional ring, provide an enhanced matrix adaptation to the teeth and a firm grip, thus reducing composite flashes and optimising stability. Replaceable, interchangeable and autoclavable, these plastic extremities are V-shaped for better positioning over the wedge.

- myTines Small**, 6 pairs – REF 6304
- myTines Medium**, 6 pairs – REF 6303
- myTines Large**, 6 pairs – REF 6306
- myTines GO**, 6 pairs – REF 6307
- myTines, assorted** (Small, Medium, Large) – REF 6310

